

# Personalized Wound Care Kit

This kit was prescribed by your physician specifically for YOU.

- Sterile Triple Helix Collagen Powder
- Sterile & Water-Resistant Bordered Gauze Dressings
- Sterile Gauze Pads



Beyond healing.



[www.quality-recovery.com](http://www.quality-recovery.com)

## Wound Care Kit Instructions

This kit was prescribed by your physician to aid in your healing journey.

### Application

### Step 1

#### Cleanse the wound


Confirm your wound(s) are clean.  
Use Sterile Gauze Pads as needed.

### Step 2

#### Dress the wound

Open Triple Helix Collagen Powder packet and apply sufficient powder, approximately ¼" depth of coverage of the wound bed.

Do not overpack or compress powder as it may expand as it absorbs fluid from the wound.

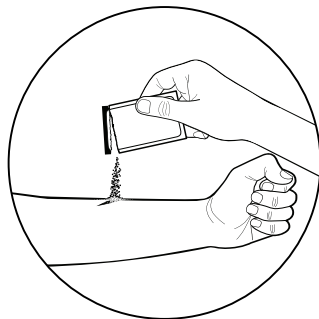

### All Contents Sterile Until Opened


STERILE R

For instructions or product questions, call (888) 391-5107. For any concerns about your wound, call your physician's office.

### Step 3

#### Apply the gauze

Apply the Sterile Bordered Gauze to secure and protect the Triple Helix Collagen Powder.

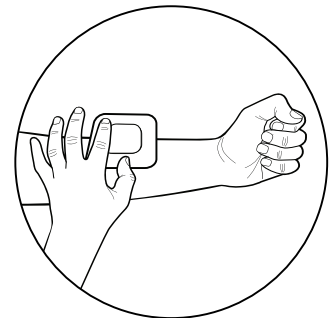

### Step 4

#### Change the dressings

Reapply Triple Helix Collagen Powder and a new Sterile Bordered Gauze daily, or as indicated by your physician. Use caution when removing the gauze.

#### Caution:

Triple Helix Collagen Powder is intended for topical use only. Avoid inhalation and flush immediately with water if product comes in contact with eyes.

#### Contraindications:

Triple Helix Collagen Powder should not be used if allergic to bovine-derived materials, or with third-degree burns or known sensitivities to dressing components.

## THANK YOU

for allowing us to be a part of your healing journey.