

Personalized Wound Care Kit **Instructions**

When to use this kit? Please follow your physician's instructions. This kit is most likely used following your original dressing protocol.

Removing the Dressing/s: Carefully remove the existing island dressing covering each wound.

Step 1 Check Your Wound/s


Confirm the wound and/or wounds are clean. Use gauze pads as needed.

Step 2 Applying Collagen Powder

Open a NEW STERILE Triple Helix Collagen Powder packet. Apply sufficient amount of powder (approximately ¼ inches depth) of coverage to the wound bed (not all collagen may be needed for your wound).


DO NOT compress powder as it may expand as it absorbs wound exudate.

If you are applying to a rounded body part, lean to the side or ask a caregiver for assistance.

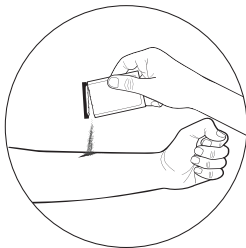

Step 3 Apply the Bordered Island Dressing

Remove the bordered island dressing from the package. Remove one side of the paper liner. Place dressing over the wound and surrounding skin. Remove the remaining liner. Carefully smooth the edge of the dressing to ensure proper skin contact. Sterile Bordered Island Dressing is shower/water resistant, not water proof.

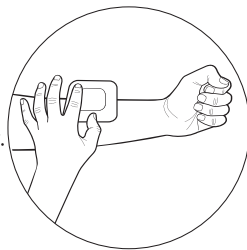

Step 4 Change the Dressing Daily

Change as directed and repeat steps 1 through 3. When changing, use caution removing bordered island dressing.