


Personalized Wound Care Kit Instructions

Step 1 Check Your Wound/s

Confirm the wound and/or wounds are clean. Use gauze pads and rubbing alcohol as needed.

Step 2 Applying Collagen Pad

Open a NEW STERILE Collagen Pad. Cut the pad diagonally to ensure proper pad length for the incision. Place collagen pad on incision, holding in place and apply the following step.

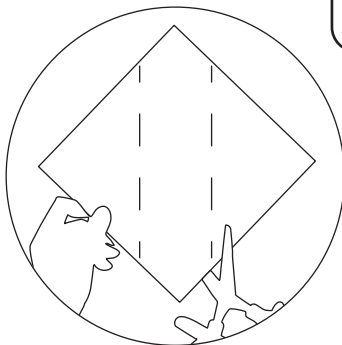

Step 3 Apply the Bordered Island Dressing

Remove the bordered island dressing from the package. Remove one side of the paper liner. Place dressing over the wound and surrounding skin. Remove the remaining liner. Carefully smooth the edge of the dressing to ensure proper skin contact.



Step 4 Change the Dressing Daily

Change as directed and repeat steps 1 through 3. When changing, use caution removing bordered island dressing. Please remove the island dressing prior to showering or bathing.

When to use this kit? Please follow your physician's instructions. This kit is most likely used following your original dressing protocol.

Removing the Dressing/s: Carefully remove the existing island dressing covering each wound.

Instructions or Product Questions

For any concerns about your wound, call your physician's office.

For detailed usage instructions and a full list of warnings, cautions, and contraindications, always refer to the manufacturer's guidelines for each product included in this kit. The instructions provided by Quality Recovery are merely suggestions on the sequence of product use. However, please note that these products have not been specifically designed or clinically evaluated for their combined use in addressing any particular clinical indication.

For instructions or product questions, call Quality Recovery at (888) 391-5107.

STERILE R All Contents Sterile Until Opened

WARNINGS:

- Due to risk of infection, discard of any product that was used and/or damaged.
- DO NOT use products if you are allergic or sensitive to materials of bovine origin.
- If excessive redness, pain, swelling or blistering occurs discontinue use of Product & Bordered Island Dressing and notify your doctor.

Thank you for allowing us to be a part of your healing journey.

Personalized 
Wound Care Kit

www.quality-recovery.com
(888) 391-5107

 **QUALITY**
RECOVERY